



HƯỚNG DẪN SỬ DỤNG

KIM LUÔN TĨNH MẠCH

Hãng sản xuất: Terumo (Philippines) Corporation

Nước sản xuất: Philippin

CẢM CHỈ ĐỊNH:

Không dùng ở khu vực có từ tính

CẢNH BÁO

- Đọc kỹ hướng dẫn sử dụng trước khi dùng
- Không sử dụng nếu bao gói sản phẩm bị hỏng hoặc bị bẩn
- Không cố gắng để luồn lại kim vào catheter khi kim đã được rút từng phần hay hoàn toàn
- Xử lý cẩn thận để tránh kim đâm vào tay
- Không sử dụng lại hoặc tiết trùng lại.
- Vứt bỏ đúng cách sau khi sử dụng tránh lây nhiễm
- **KHÔNG BẢO QUẢN TẠI NƠI CÓ ÁNH SÁNG MẠNH VÀ ĐỘ ẨM CAO. TRÁNH ÁNH SÁNG TRỰC TIẾP**

HƯỚNG DẪN SỬ DỤNG

1. Mở ống kim bằng cách vặn nhẹ phần niêm phong
2. Lấy kim luôn tra khỏi ống theo đường thẳng
3. Gò phía trên buồng chứa máu là chỉ thị hướng đầu vát của kim.
4. Đưa đầu kim vào da trước, khi đã nhìn thấy máu báo chày về buồng chứa máu thì đưa cả kim và catheter vào sâu trong ven.
5. Đưa catheter vào nằm trong ven
6. Giữ phần catheter thật chắc, rồi rút nòng kim sắt ra
7. Gắn dây truyền hoặc các thiết bị bảo vệ cần thiết

Enclosure 4:

IFU

TERUMO (PHILIPPINES) CORPORATION
SurFlo I.V Catheter Radiopaque

Contraindicated where Magnetic Resonance Imaging (MRI) is anticipated.

<PRECAUTIONS>

- **Read Instruction for Use.**
- **Do not use if the package or the product has been damaged or contaminated.**
- **Do not attempt to reinsert a partially or completely withdrawn needle.**
- **Handle with care to avoid needlestick.**
- **Do not re-use or re-sterilize**
- **Dispose of properly after single use.**
- **DO NOT STORE AT EXTREME TEMPERATURES AND HUMIDITY. AVOID DIRECT SUNLIGHT.**

<INSTRUCTIONS FOR USE>

- 1. Open the case with a simple twisting motion to break the seal.**
- 2. Remove the I.V. Catheter straight out of the case using two hands.**
- 3. A box-shaped dent located on the upper surface of the flashback chamber is a needle bevel orientation indicator.**
- 4. After confirming flashback at flashback chamber, advance the catheter and needle together a little further.**
- 5. Advance the catheter into the vein.**
- 6. Holding the catheter hub firmly, withdraw the needle.**
- 7. Connect administration set and secure the devices safely.**

CONFIDENTIAL INFORMATION