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TERUFUSION Solution Administration Set for Pediatric Use

Contents sterile, non-pyrogenic in unopened, undamaged package.

<PRECAUTIONS>

Warning

Use immediately after opening the unit package. Dispose of safely after single use to avoid risk of infection.

Precautions

- For single use only. Sterilized by gamma radiation. Do not sterilize or reuse.
- Do not use if the unit package or product has been damaged or contaminated.
- When using this set on an infusion pump, consult with the pump's manufacturer about the compatibility.
- Using the set with an infusion pump which is incapable of detecting occlusion may result in the separation of the set at its connections or damage to the set due to excessive pressure caused from occlusion.
- Do not directly touch the spike, air vent needle, venipuncture needle, or any other cannula without protector.
- The venipuncture needle included in some sets containing an air-trap connector cannot be connected to syringes, other types of infusion sets, or other devices.

<INSTRUCTION FOR USE>

Cautions

- In sets without a venipuncture needle, firmly connect a venipuncture needle or some appropriate device to the connector of the set before use.
 - When using a venipuncture needle, a winged infusion set, I.V. catheter, or an infusion pump, with this set, follow the manufacturer's instructions and precautions for using the device.
1. If a solution in a bottle-type container is to be administered, the air vent needle will be used. Hold the solution container with the outlet up.
Remove the protector from the air vent needle. Insert the air vent needle straight and the full depth into the rubber stopper of the container at the place marked with a circle. In sets not containing an air vent needle, use an appropriate air vent needle.
 2. Close the roller clamp of the solution administration set all the way.

- Remove the protector from the spike of the administration set. Insert the spike straight and the full depth into the rubber stopper of the solution container at the place marked with a circle.
3. Suspend the solution container connected to the solution administration set. Squeeze the drip chamber gently with your fingers and release it to fill it half full with solution.
 4. Open the roller clamp. Make sure that the solution has filled up to the end of the set, then close the roller clamp all the way.
 5. Make sure that the connection between the set and the venipuncture needle (or the device connected to the set) is secure. Then remove the protector from the needle by pulling it straight off. Perform venipuncture and secure the needle to the skin.
 6. Open the roller clamp to start the infusion. The roller clamp should be opened progressively and adjusted to a desired drip rate.

Drip rate: $1\text{mL} \div 60$ drops

- DO NOT STORE AT EXTREME TEMPERATURE AND HUMIDITY, AVOID DIRECT SUNLIGHT.

TERUMO CORPORATION

44-1, 2-chome, Hatagaya, Shibuya-ku, Tokyo 151-0072, Japan

Phone : 81-3-3374-8111