# Dây truyền máu TERUFUSION

## <CẢNH BÁO>

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
---

## <PRECAUTIONS>

- This device contains DEHP. Based on animal studies, significant exposure to DEHP may interfere with the normal development of the male reproductive tract. For children, pregnant and nursing women, alternative devices may be appropriate.
- Do not use if unit package is damaged.
- Do not use if protective caps are loose or displaced.
- Do not store at extreme temperature and humidity.
- Dispose of safely after single use to avoid risk of infection.
- For single use only. Do not reuse.
- Do not re-sterilize. Do not reprocess.
- Reprocessing may compromise the sterility, biocompatibility and functional integrity of the device.

## <INSTRUCTION FOR USE>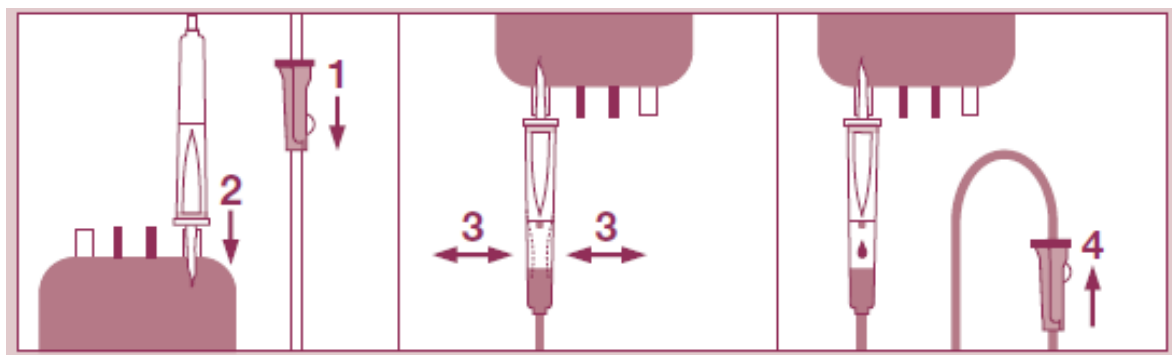

- Close the roller clamp, then remove the protector from the spike and insert it perpendicularly and to its full depth into the blood bag.
- Suspend the blood bag connected with the blood administration set. Squeeze the drip chamber slowly with your fingers to fill it about half full with blood.
- Open the roller clamp. Make sure that the blood has filled up to the end of this set, then close the clamp.
- Remove the protector from the needle by pulling straight. Perform venipuncture and secure the needle to the skin.
- Open the roller clamp and start the transfusion.

TERUMO CORPORATION

44-1, 2-chome, Hatagaya, Shibuya-ku, Tokyo 151-0072, Japan

Phone : 81-3-3374-8111