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## ENGLISH

- Sterile and non-pyrogenic in an unopened and undamaged unit package.

### <PRECAUTIONS>

- This SOLUTION ADMINISTRATION SET is for use with TERUFUSION INFUSION PUMPS. • Do not use if the unit package or the product has been damaged or soiled. • Use immediately after opening the unit packaging. Do not use if protective end caps are not in place. Dispose of safely after single use to avoid risk of infection. • **We recommend to replace the set every 24 hours, as the flow rate may deviate from the set value due to deformation of the tubing section in the pump after longer period of use.** • For single use only. Do not reuse. Do not resterilize. Do not reprocess. • Reprocessing may compromise the sterility, biocompatibility and functional integrity of the device. • **DO NOT STORE AT EXTREME TEMPERATURES AND HUMIDITY. AVOID DIRECT SUNLIGHT.**

### <INSTRUCTIONS FOR USE>

#### CAUTIONS

- Before carrying out the steps described below, firmly attach a venipuncture needle or some appropriate device to the connector of the set.
  - When using a device, e.g. a venipuncture needle, a winged infusion set, an IV catheter, CV catheter and infusion pump in combination with this set, follow the manufacturer's instructions and precautions for use for these devices.
1. If a solution in a bottle-type container is to be administered, hold the container with its outlet up, remove the protector from the airway needle, and insert the airway needle perpendicularly and to its full depth into the rubber stopper of the container at the position marked with a circle. If the set is supplied without an airway needle, use an appropriate airway needle.
  2. Close the roller clamp, then remove the protector from the spike and insert it perpendicularly and to its full depth into the rubber stopper of the solution container at the point marked with a circle.

#### CAUTION

Do not insert the plastic spike of the set slantwise against the rubber stopper, or give a sideways force, to prevent the spike from bending or breaking.

3. Suspend the solution container connected with the solution administration set for pump. Squeeze the drip chamber slowly with your fingers to fill it about half full with the solution.
4. Open the roller clamp. Make sure that the solution has filled up to the end of this set, then close the clamp and position the tubing into the infusion pump.
5. Make sure that the venipuncture needle, or the appropriate device is properly connected to the set. Remove the protector from the needle by pulling straight. Perform venipuncture and secure the needle to the skin.
6. Open the roller clamp and start the pump.

**TERUMO**


TERUFUSION®

## SOLUTION ADMINISTRATION SET for infusion pump

NECESSAIRE A PERFUSION pour pompes / INFUSIONSGERÄT für Infusionspumpen  
EQUIPO DE INFUSION para usar con bombas / ADMINISTRATIONSSÆT til infusionspumpe  
INFUSSET voor gebruik met pompen / INFUSIONSSAGGREGAT för pump  
SET PER INFUSIONE DI SOLUZIONI con pompa / EQUIPO DE INFUSÃO INTRAVENOSA para uso em bomba de infusão

20 drops per milliliter / 20 Gouttes par millilitre /  
20 Tropfen pro Milliliter / 20 Gotas por mililitro /  
20 Dråber pr. milliliter / 20 Droppar per milliliter /  
20 droppar per milliliter / 20 Gocce per millilitro /  
20 Gotas por mililitro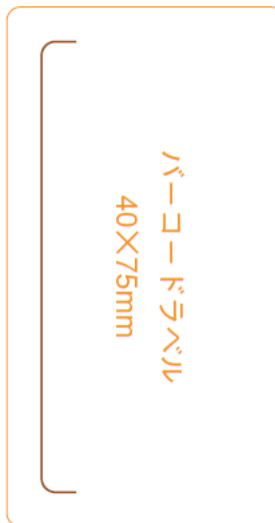

**20**

UNITS / UNITES  
STÜCK / UNIDADES  
STK / STUKS / STYCK  
UNITA / UNIDADES