

色指定

1 色

黑 black

色表示的文字、线框等内容不用印刷。

放大倍率(scale) : 100%

说明书

210mm × 297mm

2014-12 INSTRUCTIONS FOR USE

Terumo Maintenance Code



SF27A320C1

SAFEED[®] EXTENSION TUBE

CE
0123

ET


This product is a disposable EXTENSION TUBE made mainly from PVC、ABS material. It has excellent elasticity and plasticity、extreme smoothness and high transparency.

[ITEM CODE NUMBER LIST]

SF*ET2022	SF*ET0525	SF*ET1725	SF*ET3825
SF*ET2527	SF*ET5527	SF*ET1020	SF*ET3520
SF*ET2022L	SF*ET0525L	SF*ET1725L	SF*ET3825L
SF*ET2527L	SF*ET5527L	SF*ET1020L22	SF*ET3520L22
SF*ET2022L22	SF*ET0727L22	SF*ET152EL22	SF*ET2527L22
SF*ET152E	SF*ET1725L22	SF*ET5527L22	SF*ET0222L22

[MAIN STRUCTURE]

This product mainly consists of male conical fittings or lock fittings, tube, female conical lock fittings, conic cap or protector, lock cap; the sketch layout are as follows:

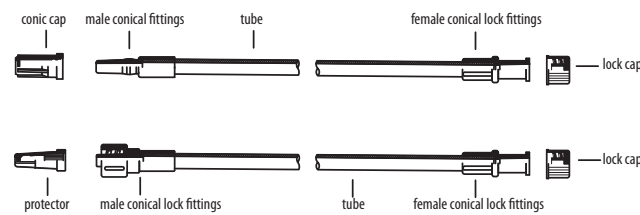

[INTENDED USE]

This product is mainly use for extending infusion flow path during intravenous anaesthesia or infusion.

[PRECAUTIONS]

- Do not use for blood transfusion.
- Please avoid the solution from sticking to the surface of conical fittings, or the joints may be loose.
- To avoid cracks in the conical fittings, please be guided as follows:
 - a. Please do not use medicament containing alcohol to disinfect the joints;
 - b. Please do not use medicament containing oily contrast medium or alcohol.
- Please do not use the product with high pressure injection line (e.g. blood vessel contrast medium) for it may cause damage.

- This product is intended for single use, do not resterilize or reuse.
- Do not use if the product is contaminated or damaged; do not use if the package is compromised.
- Use immediately after opening the unit package, dispose of after use.

[INSTRUCTIONS FOR USE]

1. Remove the SAFEED EXTENSION TUBE from its package in an aseptic manner.
2. Secure the connection to the luer tip connector.

[EXPIRATION DATE]

- The period of validity of this product is 5 years.

[STORAGE]

**Do not store at extreme temperatures and humidity.
Avoid direct sunlight.**

TERUMO MEDICAL PRODUCTS (HANGZHOU) CO., LTD.
M4-9-5 Hangzhou Economic and Technological Development Zone, 310018 Hangzhou, P.R.CHINA
TEL : +86-571-87318100

EC REP TERUMO EUROPE N.V., Interleuvenlaan 40, 3001 LEUVEN, BELGIUM

AU Sponsor TERUMO Australia Pty Ltd
Macquarie Park NSW 2113 Australia T: +61 2 9878 5122
E: cs_australia@terumo.co.jp

Do not reuse

Do not use if package is damaged

Consult instructions for use

Non-pyrogenic

Not made with DEHP

Sterile

Sterilized using ethylene oxide

Batch Code

Use-by date

Manufacturer

Authorized representative in the European Community

TERUMO[®]

TERUMO MEDICAL PRODUCTS (HANGZHOU) CO., LTD.

® : Registered Trademark

© TERUMO CORPORATION 12/2014

版下管理编号: SF_SF27A320C1_E_50_002