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### IMUGARD III-PL

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Rev. 2013-01 N-IP2002N

**IMUGARD III-PL**  
4B

Leukocyte removal filter for preparation of leukocyte-poor platelets during transfusion. For Platelet Concentrates or Single Donor Platelets.

For bedside use. For use at the patient's bedside. For use at the patient's bedside. For use at the patient's bedside.

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ENGLISH

**DESCRIPTION / DESIGNATION**

This device is indicated for the removal of leukocytes from a pool of up to 10 units of platelet concentrates or from one unit of Single Donor Platelets (apheresis).

• Sterilized by ethylene oxide. Sterile in an unopened and undamaged unit package.

• Non-pyrogenic fluid path.

**PRECAUTIONS**

This device contains DEHP. Based on animal studies, significant exposure to DEHP may interfere with the normal development of the male reproductive tract. For children, pregnant and nursing women, alternative devices may be appropriate.

• Do not use if the unit package or the product has been damaged or soiled.

• Dispose of safely after single use to avoid risk of infection.

• Do not store at extreme temperatures and humidity. Avoid direct sunlight.

• Read the following instructions carefully before using.

• For single use only. Do not reuse. Do not resterilize. Do not reprocess. Reprocessing may compromise the sterility, biocompatibility and functional integrity of the device.

**INSTRUCTIONS FOR USE**

**CAUTION** • Strict aseptic technique should be observed at all times.

**Attach platelet bag**

- Close the YELLOW "C" clamp.
- Move the roller clamp to just below the drip chamber and close it.
- Open the peel tab of the outlet port on the unit of platelets to be filtered.
- Remove the protector from the spike located at the inlet end of the filter tubing and insert the spike into the opened outlet port on the unit of platelets to be filtered.

**CAUTION** Do not hold the tubing part of the spike while inserting it into the outlet port. Instead, hold the hub part of the spike firmly. This will prevent loosening of tubing part from the hub resulting in leakage. (Fig. 1)

- Hang the unit of platelets to be filtered on an I.V. stand.
- Attach the venipuncture needle to the Luer lock connector located at the end of the tubing (leave the needle cap on the needle).

**Prime filter**

- To start filtration, invert the filter and drip chamber and open the YELLOW "C" clamp. Open the roller clamp which is located below the drip chamber. (Fig. 2)

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