SA17EA002C

2020-10

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SURPLUG AD Manifold

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PVC Free

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SURPLUG AD Manifold

Index of Symbols

Does not contain PVC / drug contact component

Does not contain latex

Non-pyrogenic

Do not use if package is damaged

Consult instructions for use

Do not reuse

Manufacturer

Sterilized using ethylene oxide

Intended Use

SURPLUG AD Manifold is used as a valve port connected to the fluid infusion set or a similar device for injection. It can also be used for blood sampling and transfusion. SURPLUG AD Manifold has the flow pathway to prevent medications from stagnation.

Warnings

- Disinfect the SURPLUG AD Manifold valve port before use.
- To apply heparin lock, close the clamp between the catheter and SURPLUG AD Manifold before removing the syringe. While applying heparin, do not open the clamp that has been closed.

Cautions

- For single use only. Do not reuse.
- Do not resterilize.
- Do not use a needle for injection.
- Do not apply heparin lock by connecting directly to an IV catheter or CV catheter without a clamp or via an extension tube without a clamp.
- Do not use if the package is damaged.

Precautions

- Make sure that there are no loose connections before use.
- When connecting a male luer connector to the SURPLUG AD Manifold valve port, be sure to insert and remove it straight. Do not connect at an angle or with excessive twisting.
- Do not use if any part of the product is damaged.
- Make sure regularly that there are no breakage, no loose connections or leakage during use.
- Do not apply high-pressure injection such as contrast agent using injector.
- Do not connect a connector with sharp edge to the SURPLUG AD Manifold valve port.
- When using a connector with bump around the end, a connector with sharp edge or a connector with burr on the edge, make sure that the flow pathway is secured before starting injection, because the SURPLUG AD Manifold valve port may remain closed.
- When performing a rapid infusion or absorption from an injection port, take measures blocking other infusion sets or extension tubes connected to other injection ports to prevent from the backflow.
- Be careful that the back check valve does not completely prevent from the backflow.

Storage Method

- Do not store in direct sunlight and high temperature and humidity.

Expiry Date

- Expiration date stated in the outer box.

TERUMO

TERUMO MEDICAL PRODUCTS (HANGZHOU) CO., LTD.
M4-9-5 Hangzhou Economic and Technological Development Zone, 310018 Hangzhou, P.R.CHINA

All brand names are trademarks or registered trademarks of TERUMO CORPORATION and their respective owners.

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© : TERUMO CORPORATION 10/2020