TERUMO

TERUFUSION™

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TERUMO MEDICAL PRODUCTS (HANGZHOU) CO., LTD.

M4-9-5 Hangzhou Economic and Technological Development Zone, 310018 Hangzhou, P.R.CHINA

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STERILEEO

For TERUFUSION PUMPs

BLOOD ADMINISTRATION SET for infusion pump

Index of Symbols

Do not use if package is damaged

P

Pressure

Drops per milliliter

STERILEEO

Sterilized using ethylene oxide

~~LATEX~~

Does not contain latex

Content

Non-pyrogenic


Do not reuse

INTENDED USE

The product is intended for use in infusing blood volumetrically by using an infusion pump and is ready for general gravity use and to be discarded after single use

INSTRUCTION FOR USE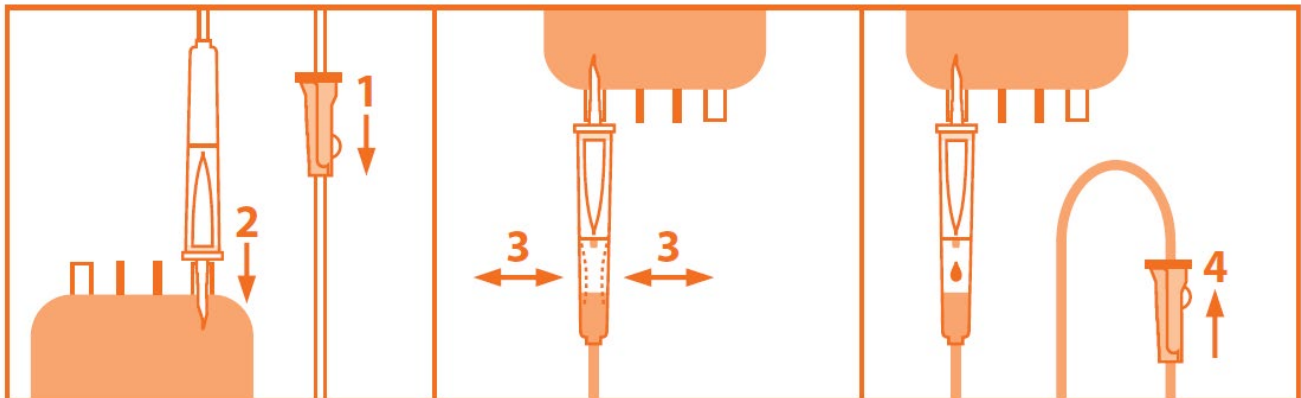

1. Close the Flow regulator first.
2. Then insert the spike (1 way or 2 way) which has removed the Protective cap into the blood bag port.
3. In order to let the blood flow into the drip chamber, squeeze the drip chamber first.
4. Open the Flow regulator
5. Finally, open the cap of air filter, to start the blood infusion. (for the reason of TB*PU300SE&TB*PU300SYE without air filter, so leave out the steps of open the cap)

And if the TERUFUSION BLOOD ADMINISTRATION SET for infusion pump with a AFF clamp, the operation procedure shall be like this.

1. Make sure that the AFF clamp is not closed.
2. Slide the AFF clamp down.
3. Make sure that the tube is positioned in the AFF clamp as indicated in Fig1 A, and insert the clamp into the dedicated aperture of the infusion pump.
4. Then attach the tube properly to the tube guide slots, close the pump door to give a infusion after you have set properly

Caution: Read the instruction manual of the infusion pump before use.

CAUTIONS

- For single use only.
- Do not reuse.
- Do not resterilize.
- Do not reprocess.
- Reprocessing may compromise the sterility, biocompatibility and functional integrity of the device.

PRECAUTIONS

- Do not use if protective caps are loose or displaced.
- Use immediately after opening the unit packaging.
- Dispose of safely after use to avoid risk of infection.
- Ensure that all connections made with the Terufusion blood administration set are secure.
- Do not turn upside down or shake the drip chamber during administration.
- Disinfect the injection port before use.
- We recommend replace the set every 24 hours, as the flow rate may deviate from the set value due to deformation of the tubing section in the pump after long period of use.

STORAGE METHOD

DO NOT STORE AT EXTREME TEMPERATURES.

TERUMO MEDICAL PRODUCTS (HANGZHOU) CO., LTD.
M4-9-5 Hangzhou Economic and Technological Development Zone, 310018 Hangzhou, P.R.CHINA
MADE IN CHINA

AU Sponsor **Terumo Australia Pty Ltd** Macquarie Park NSW 2113 Australia T: +61 2 98785122

All brand names are trademarks or registered trademarks of TERUMO CORPORATION and their respective owners.