\*MS69EA500A\*

2018-01

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TERUMO

TERUMO CORPORATION

44-1, 2-chome, Hatagaya, Shibuya-ku, Tokyo 151-0072, JAPAN MADE IN JAPAN

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\*MS69EA500A\*

2018-01

# MEDISAFE® FINETOUGH® LANCING DEVICE

This product is intended for capillary blood collection from the fingertip, earlobe or heel using a disposable Lancet for FINETOUGH. Please read carefully before use.


### Caution

- Before pricking, keep selected puncture site clean or disinfect by using an alcohol swab.
- Make sure that blood or devices contaminated with blood, including tissue paper and other items you have used, do not come into contact with any other person.
- Keep away from children.
- Do not store the FINETOUGH Lancing Device with the lancet attached. It can cause malfunction of the lancing depth adjustment.

### With regard to Lancet for FINETOUGH, please be careful of the following precautions.


- For single use only. Do not reuse. Do not resterilize. Do not reprocess.
- Reprocessing may compromise the sterility, biocompatibility and functional integrity of the device.

### Components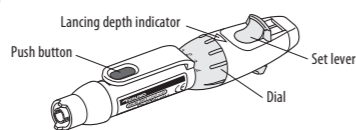


### Instructions for use

- Before pricking, keep selected puncture site clean or disinfect by using an alcohol swab.
- Turn the Dial to select the desired lancing depth. The FINETOUGH Lancing Device allows you to choose between 5 levels of lancing depth. (Fig.1)
  - Soft or thin skin (shallow lancing depth) ..... 1
  - Normal skin ..... 2, 3
  - Hard or thick skin (deep lancing depth) ..... 4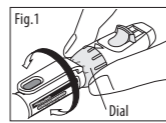


### Caution in Use

- Do not set the Lancing depth indicator between two depth markers.
- Make sure that the Lancing depth indicator is pointing at a Scale-bar on the Dial. (Fig.2)
- Start with the shallow lancing depth to find the appropriate lancing depth.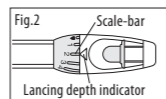



- Insert the lancet into the FINETOUGH Lancing Device. Twist and pull the lancet cap to remove it from the lancet. (Fig.3)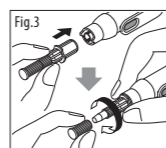


### Caution in Use

- Only use Lancet for FINETOUGH.
- Do not use the lancet when the packaging is damaged, when the lancet cap is previously removed, when the lancet is damaged or any defect is found.

- When you insert the lancet into the FINETOUGH Lancing Device, be careful about the direction of the lancet and insert it straight.
- Insert the lancet as far as it will go until you hear a clear "Click"; make sure not to push the Push button. (Fig.4)
- If the Push button is pushed accidentally, keep the lancet attached and pull the Set lever until you hear "Click". (Fig.5)
- Once the lancet is detached from the FINETOUGH Lancing Device, do not reinsert it. If you reinsert the lancet, the needle will remain exposed, without the "Click". Replace detached lancet with a new one.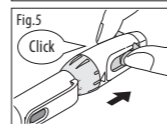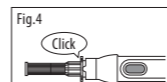

- Place the lancet gently against the selected puncture site and push the Push button. (Fig.6)
- After pricking, gently squeeze around the puncture site to obtain a drop of blood.

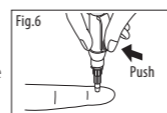### Caution in Use

- Blood will clot when it is left in the air. Immediately after taking the blood sample, proceed with blood testing.
- After using the blood sample, wipe the puncture site with a dry and clean tissue paper. Stop bleeding (for example by using sterile adhesive tape).

- Remove the used lancet straight from the FINETOUGH Lancing Device. Please dispose of used lancets and contaminated materials appropriately to a special container immediately after single use in order to avoid accidental needle-stick injuries or contamination to others.

### Caution in Use

- The needle may come out from the tip of the lancet if there is a strong impact on the lancet before disposal.
- If product fails to operate correctly, please stop using and replace the FINETOUGH Lancing Device.
- If the FINETOUGH Lancing Device is deformed or damaged, stop using and replace the FINETOUGH Lancing Device.

### Cleaning instructions

Clean the FINETOUGH Lancing Device as required using a mild liquid cleaner or a soft tissue moistened with water.

In addition, the FINETOUGH Lancing Device can be disinfected by wiping the surface of the device with ethanol, isopropyl alcohol or chlorhexidine gluconate according to the instructions of the disinfectants manufacturer. Do not immerse the FINETOUGH Lancing Device completely in liquid.

### Storage conditions

- Avoid exposure to water and direct sunlight.
- Do not store at extreme temperatures and humidity.

### Package

1 unit / box

Lancet  
0197

Lancing Device

TERUMO CORPORATION  
44-1, 2-chome, Hatagaya, Shibuya-ku, Tokyo 151-0072, JAPAN MADE IN JAPAN  
TERUMO EUROPE N.V., Interleuvenlaan 40, 3001 LEUVEN, BELGIUM

TERUMO is a trademark of TERUMO CORPORATION. ©: Registered Trademark ©TERUMO CORPORATION 2018 17K20

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TERUMO is a trademark of TERUMO CORPORATION. ©: علامة تجارية مسجلة TERUMO CORPORATION 2018 17K20