

HƯỚNG DẪN SỬ DỤNG




BƠM TIÊM NHỰA KHÔNG KIM
Hãng sản xuất: Terumo (Philippine) Corporation
Nước sản xuất: Philippin

CẢNH BÁO

- Sử dụng ngay sau khi xé bao gói sản phẩm
- Sau khi sử dụng, vứt bỏ đúng cách và an toàn để tránh nguy cơ lây nhiễm.
- Xử lý cẩn thận để tránh kim đâm vào tay
- Không thay vỏ bọc sau khi sử dụng.

HƯỚNG DẪN SỬ DỤNG

- Để mở bao gói sản phẩm: gỡ lớp bao bì trên của bao gói (hình 1)

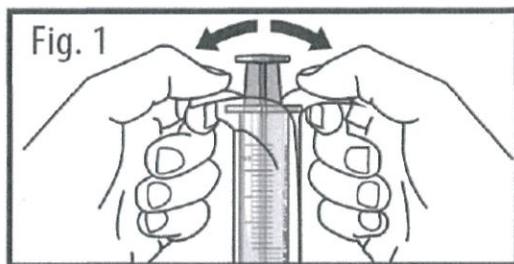

CÁC BIỆN PHÁP PHÒNG NGỪA

- Sản phẩm được khử trùng bằng chùm electron. Không tiệt trùng lại hoặc tái sử dụng.
- Không sử dụng nếu bao gói hoặc sản phẩm đã bị hư hỏng hoặc bị ô nhiễm.

KHÔNG ĐỂ SẢN PHẨM Ở NƠI CÓ ĐỘ ẨM VÀ NHIỆT ĐỘ CAO
TRÁNH TRỰC TIẾP DƯỚI ÁNH NẮNG MẶT TRỜI

Sử dụng một lần

Tiệt trùng - Không độc - Không có chất gây sốt - Không LATEX

TERUMO (PHILIPPINES) CORPORATION
Terumo Syringe without Needle

INSTRUCTIONS FOR USE

<WARNINGS>

- Use immediately after opening unit package.
After use, dispose of properly and safely to eliminate risk of infection.
- Handle with care to avoid needle sticks.
Do not replace needle sheath after use.

<INSTRUCTIONS FOR USE>

To open blister package: peel back top layer of packaging (Fig. 1)



<PRECAUTIONS>

- Sterilized by electron beam. Do not resterilize or reuse.
- Do not use if the unit package or product has been damaged or contaminated.
- Do not use for high pressure injection of contrast media.
- DO NOT STORE AT EXTREME TEMPERATURE AND HUMIDITY. AVOID DIRECT SUNLIGHT.

SINGLE USE . STERILE . NON-TOXIC . NON-PYROGENIC . LATEX FREE.