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#### Mô tả/Thiết kế

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©TERUMO CORPORATION October 2009 09J26

**ENGLISH****TERUMO SAMPLING SITE COUPLER****DESCRIPTION / DESIGNATION**

- The TERUMO SAMPLING SITE COUPLER is intended to take blood samples or blood component samples from a TERUFLEX Blood Bag or a Transfer Bag or from an equivalent blood container.
- After sampling, blood and blood components shall be treated following local regulations for transfusion.
- Sterilised by ethylene oxide. Sterile in an unopened and undamaged unit package.
- Non-pyrogenic fluid path.

**PRECAUTIONS**

- Dispose of safely after single use to avoid risk of infection.
- Do not use if the package or the product has been damaged or soiled.
- Do not store at extreme temperatures and humidity. Avoid direct sunlight.
- Read the following instructions carefully before use.
- For single use only. Do not reuse. Do not resterilize. Do not reprocess. Reprocessing may compromise the sterility, biocompatibility and functional integrity of the device.

**INSTRUCTIONS FOR USE**

1. Use an aseptic technique.
2. Remove the SAMPLING SITE COUPLER from its packaging.
3. Open the outlet port of the blood bag or transfer bag.
4. Remove the protector from coupler by twisting and insert the spike of the coupler completely into the outlet port.
5. Remove the cap of the sampling port and disinfect rubber surface of SAMPLING SITE COUPLER, then insert the needle and take the sample.

**CAUTION** DO NOT REMOVE SAMPLING SITE COUPLER FROM THE BLOOD BAG OR TRANSFER BAG.

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