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BỘ TÚI MÁU RỘNG (TERUMO TRANSFER BAG FOR ALIQUOT)

MÔ TẢ/CHỈ ĐỊNH

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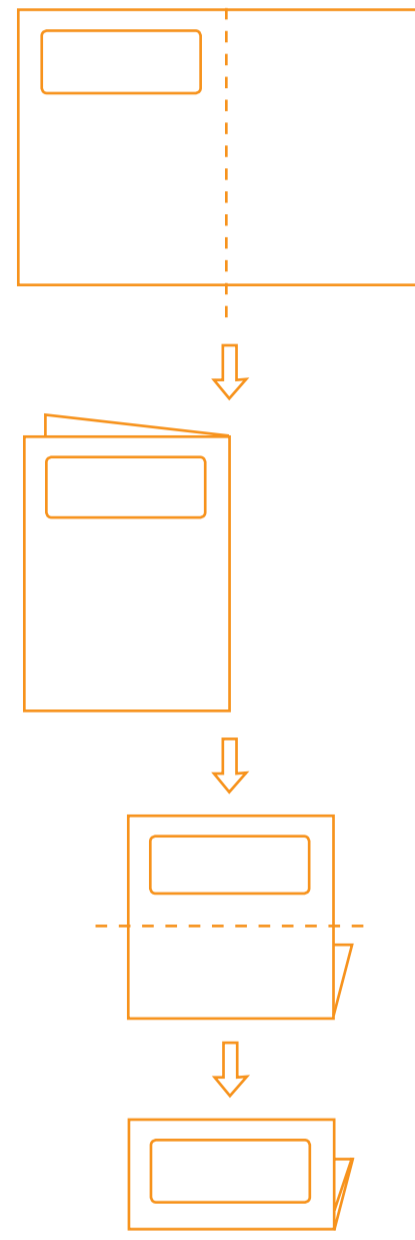
Icons and symbols for safety and handling: Non-pyrogenic fluid path, Do not vent, Consult instructions for use, Do not reuse, STERILE EO, DEHP, PHT.

®: Registered Trademark ®: Marque Enregistrée ®: Eingetragenes Warenzeichen ®: Marca Registrada ®: Marchio Registrato ®: Geregistreerd Handelsmerk ®: Registerat Varumärke

TERUMO CORPORATION 44-1, 2-CHOME, HATAGAYA, SHIBUYA-KU, TOKYO 151-0072, JAPAN
TERUMO EUROPE N.V. INTERLEUVENLAAN 40, 3001 LEUVEN, BELGIUM
MADE IN JAPAN © TERUMO CORPORATION October 2009 09J22

ENGLISH

TERUFLEX® TRANSFER BAG

DESCRIPTION/DESIGNATION

- Intended for collection, preservation and transfusion of blood components.
Sterilised by ethylene oxide. Sterile in an unopened and undamaged unit package.
Non-pyrogenic fluid path.

PRECAUTIONS

- This device contains DEHP. Based on animal studies, significant exposure to DEHP may interfere with the normal development of the male reproductive tract.
Do not use if the packaging or the product has been damaged or soiled.
Do not store at extreme temperatures and humidity. Avoid direct sunlight.
Read the following instructions carefully before use.
For single use only. Do not reuse. Do not resterilize. Do not reprocess. Reprocessing may compromise the sterility, biocompatibility and functional integrity of the device.

INSTRUCTIONS FOR USE

- CAUTION A strict aseptic technique should be observed at all times. Do not vent.
PRECAUTION When using CLOSED SYSTEM PROCESSING, make sure that the tube of the blood bag to be connected has the same inner diameter as

the one of the transfer bag, and fits within the specifications of the manufacturer of the sterile tubing welder. Connect both tubes following the instructions for use of the device and proceed with #5.

- 1. Place a plastic clamp on the tube of the transfer bag. (In case of double spikes, clamp both tubes between the Y connection and the spike.)
2. Prepare the outlet port of the blood bag to be connected.
3. Remove the protector from the spike and insert it into the outlet port, using a twisting motion.

CAUTION Do not touch the spike during and after removal of its protector.

- 4. Remove the clamp to allow blood component transfer to the transfer bag.
5. Upon completion of the transfer, seal the tube at the desired position (in case of closed system processing: between the weld and the transfer bag) and cut it.

CAUTION Do not remove the spike from the blood bag unit. Dispose of safely following the local procedures for the disposal of medical waste.

- 6. To collect the blood component through the second spike, repeat steps #2 to #5.

CAUTION Refer to the local regulations regarding the storage of blood components.

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TERUMO TRANSFER BAG FOR ALIQUOT

DESCRIPTION/DESIGNATION

- For single use. Sterile and non-pyrogenic. Sterilised by ethylene oxide gas.
- Intended collection, preservation and transfusion of blood components.

PRECAUTIONS

- Do not use if the packaging or the product has been damaged or soiled.
- Do not store at extreme temperatures and humidity. Avoid direct sunlight.
- Read the following instructions carefully before use.

INSTRUCTIONS FOR USE

CAUTION A strict aseptic technique should be observed at all times. Do not vent.

PRECAUTION When using **CLOSED SYSTEM PROCESSING**, make sure that the tube of the blood bag to be connected has the same inner diameter as the one of the transfer bag, and fits within the specifications of the manufacturer of the sterile tubing welder.

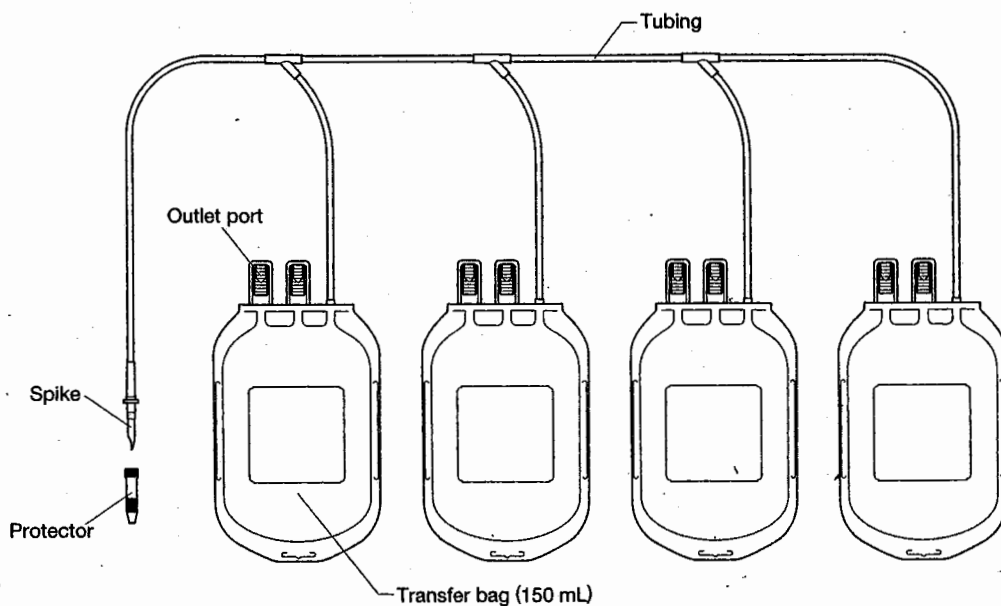
Connect both tubes following the instructions for use of the device and proceed with #5.

1. Place a plastic clamp on the tube of the transfer bag.
2. Prepare the outlet port of the blood bag to be connected.
3. Remove the protector from the spike and insert it into the outlet port, using a twisting motion.

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4. Remove the clamp to allow blood component transfer to the transfer bag.
5. Upon completion of the transfer, seal the tube at the desired position (In case of closed system processing: between the weld and the transfer bag) and cut it.

- CAUTIONS**
- Do not remove the spike from the blood bag unit. Dispose of safely following the local procedures for the disposal of medical waste.
 - Refer to local regulations regarding the storage of blood components.

TERUMO®

TERUMO CORPORATION
TOKYO, JAPAN.
MADE IN JAPAN