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MS69E812

MEDISAFE FIT® Control Solution

For MEDISAFE FIT Brand Blood Glucose Meters for self-testing

Instructions for use

Precautions on use

- Do not swallow MEDISAFE FIT Control Solution (hereinafter called "the product" or "this product") or apply it on the eyes.
- If the product has been in contact with your eyes or mouth, immediately wash off the product with water. If you feel anything abnormal, consult a physician.
- If the product has been in contact with your hands or clothes, wash well with water.
- Keep away from children.
- Avoid direct sunlight.
- Store the product at 1-30°C. If the product is stored in a refrigerator or the like, let it acclimate to room temperature (20-30°C) before using it.
- Do not freeze the product.
- If the product has been kept sealed, the expiration date is 1 year after the manufacturing date. When the expiration date printed on the box and bottle label has passed, do not use the product even if it has been kept sealed.
- Write the opening date on the bottle label. Use the product within 3 months after opening. However, if the expiration date printed on the box and bottle label is earlier than 3 months after the opening date, use the product by the expiration date.
- This product is for use with the MEDISAFE FIT Brand Blood Glucose Meters and MEDISAFE FIT Blood Glucose Test TIP.
- When used in high altitude areas (e.g., in mountainous areas), there may be some differences in the test results.

Summary and explanation

This product is an aqueous solution containing glucose. It is used to confirm that the MEDISAFE FIT Brand Blood Glucose Meters and MEDISAFE FIT Blood Glucose Test TIP are functioning appropriately. When used for the MEDISAFE FIT Brand Blood Glucose Meters, use the product in combination with a designated blood glucose test tip (MEDISAFE FIT Blood Glucose Test TIP).

This product is available in 3 concentration levels (L: about 40 mg/dL, M: about 120 mg/dL, H: about 300 mg/dL). Use the Control Solution with the concentration as advised by your physician. Before the product is used, carefully read not only the instructions for this product but also the instructions of the MEDISAFE FIT Brand Blood Glucose Meters and MEDISAFE FIT Blood Glucose Test TIP.

Content

MEDISAFE FIT Control Solution 4 mL/bottle
Concentration (L, M, H) is printed on the box and bottle label.

Composition

This product is an aqueous solution containing glucose and sodium carboxymethylcellulose.

How to Use

When using a new bottle of Control Solution, write the opening date on the bottle label. The expiration date is 3 months after opening. Before use, confirm that 3 months have not passed since opening or that the expiration date printed on the bottle label has not passed. Perform tests at 20-30°C.

1. Set the Meter to Control Solution testing mode.
2. Peel off the film-seal of the TIP case completely, and attach the Test TIP firmly on the Meter. Then, pull the case off straight. Check the Meter is ready to test.
3. Put one drop of the Control Solution on a film-seal (the silver side). The volume of a drop may be about 5-6mm in diameter. When the leading edge of the Test TIP is applied to the droplet, the Control Solution is drawn up and the countdown starts immediately. After the countdown starts, immediately move the Test TIP away from the droplet and put the Meter aside gently.
4. After the countdown, the test result with time, confirm that the result is within the acceptable range for the Control Solution with a given concentration printed on the sticker label on the Test TIP box.
5. When discarding the Test TIP, cover it with an empty TIP case, while the Test TIP is still attached to the Meter. Slide forward the eject lever and push off the used Test TIP into the TIP case.
6. If the result is not within the acceptable range, repeat the step from 1 to 5 with a new Test TIP to perform a re-test.
7. If testing is to be performed using the Control Solution with a different concentration, repeat the step from 1 to 5 with a new Test TIP.
8. Test result would be referred in a record.
9. When the testing is completed, turn the power off.
10. After using the Control Solution, wipe the bottle nozzle with clean tissue paper or the like, and store it with the cap on tight. Do not leave the bottle nozzle standing without a cap.

Other precautions

- Test at the timing specified in the instructions for the MEDISAFE FIT Brand Blood Glucose Meters.
- Do not directly apply the Control Solution from the bottle nozzle to the Test TIP nozzle.
- Start testing immediately after the Control Solution has been dripped on the film-seal.
- This product can be used for about 60 sessions of testing. If the remaining volume is very small, use a new bottle.
- Comply with the waste disposal rules in your local community or at a given facility when discarding the product.

Test Results

If the result is not within the acceptable range for the Control Solution with a given concentration printed on the sticker label on the Test TIP box, perform the test again with a new Test TIP. For the re-test, drip the new Control Solution.
If the re-test also give the indication "FAILED", it may be due to the causes listed below.

Cause	Countermeasure
Testing was not correctly performed.	Perform testing in accordance with the instructions for this product, the instructions for the MEDISAFE FIT Brand Blood Glucose Meters and the instructions for MEDISAFE FIT Blood Glucose Test TIP.
The expiration date for the Control Solution has passed or 3 months have passed since opening the bottle.	Use a new Control Solution.
There is a problem with storage of the Control Solution.	Use a new Control Solution. Store it correctly in compliance with the instructions given in this document.
There is a problem with MEDISAFE FIT Blood Glucose Test TIP or the MEDISAFE FIT Brand Blood Glucose Meters.	Refer to troubleshooting. If the problem remains unsolved of the MEDISAFE FIT Brand Blood Glucose Meters, contact TERUMO.

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© - Registered Trademark