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\*SA17EA004C\*

2020-10

# SURPLUG AD Extension tube

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# SURPLUG AD Extension tube

### Index of Symbols

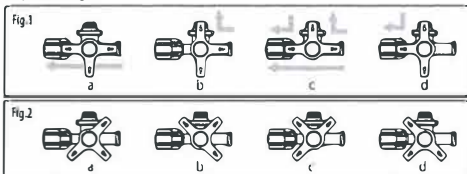
<b>PVC Free</b> Does not contain PVC / drug contact component	<b>LATEX Free</b> Does not contain latex	<b>Non-pyrogenic</b>	<b>Do not use if package is damaged</b>
<b>Consult instructions for use</b>	<b>Do not reuse</b>	<b>Manufacturer</b>	<b>Sterilized using ethylene oxide</b>
<b>SURPLUG AD injection port (T-type)</b>	<b>SURPLUG AD Three-way stopcock</b>	<b>SURPLUG AD injection port (I-type)</b>	<b>Female Connector (Open type)</b>
<b>Back check valve</b>	<b>Micro Filter (0.2µm)</b>	<b>Lock Connector</b>	<b>Slip Connector</b>

### Intended Use

SURPLUG AD Extension tube is used as a valve port connected to the fluid infusion set or a similar device for injection. It can also be used for blood sampling and transfusion. SURPLUG AD Extension tube has the flow pathway to prevent medications from stagnation.

### How to Use Extension Tube with SURPLUG AD Three-way stopcock

- The arrows below show flow pathway. To set stopcock position depending on use. By switching stopcock position, flow pathway is switched.
- To keep the stopcock position as Fig.1c to prevent medications from stagnation in the flow pathway, even if the SURPLUG AD Extension tube valve port is not used.
- In order to close the flow path in all directions, it is set at the position of the stopcock in Fig.2.

### Warnings

- Disinfect the SURPLUG AD Extension tube valve port before use.
- To apply heparin lock, close the clamp or the stopcock between the catheter and SURPLUG AD Extension tube before removing the syringe. While applying heparin, do not open the clamp or the stopcock that has been closed.

### Cautions

- For single use only. Do not reuse.
- Do not resterilize.
- Do not use a needle for injection.
- Do not use if the package is damaged.

### Precautions

- Make sure that there are no loose connections before use.
- Set the stopcock into open position and connect to the SURPLUG AD Extension tube valve port (Fig.1b/Fig.1c/Fig.1d).
- When connecting a male luer connector to the SURPLUG AD Extension tube valve port, be sure to insert and remove it straight. Do not connect at an angle or with excessive twisting.
- Do not use if any part of the product is damaged.
- Make sure that you do not damage the tube by pinching it or using any other sharp objects.
- Do not apply excessive force to the tube or connecting part of the tube, by pulling, pushing or bending the tube.
- Make sure regularly that there are no breakage, no loose connections or leakage during use.
- Do not apply high-pressure injection such as contrast agent using injector.
- Do not connect a connector with sharp edge to the SURPLUG AD Extension tube valve port.
- When using a connector with bump around the end, a connector with sharp edge or a connector with burr on the edge, make sure that the flow pathway is secured before starting injection, because the SURPLUG AD Extension tube valve port may remain closed.
- When performing a rapid infusion or absorption from an injection port, take measures blocking other infusion sets or extension tubes connected to other injection ports to prevent from the backflow.
- Be careful that the back check valve does not completely prevent from the backflow.

### Storage Method

- Do not store in direct sunlight and high temperature and humidity.

### Expiry Date

- Expiration date stated in the outer box.

TERUMO

TERUMO MEDICAL PRODUCTS (HANGZHOU) CO., LTD.  
M4-9-5 Hangzhou Economic and Technological Development Zone, 310018 Hangzhou, P.R.CHINA

All brand names are trademarks or registered trademarks of TERUMO CORPORATION and their respective owners.

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©: TERUMO CORPORATION 10/2020