\*TS69J001C2\*

 **TERUMO**

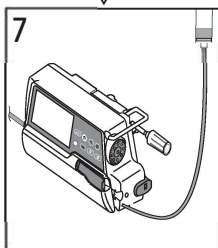
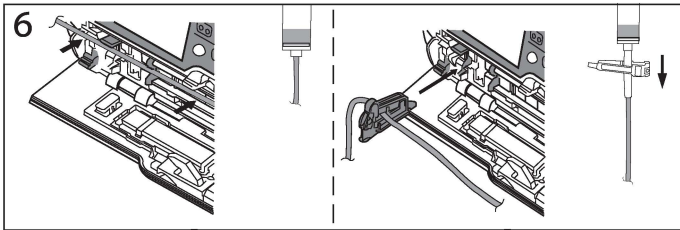
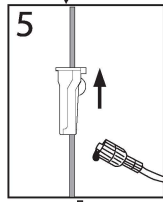
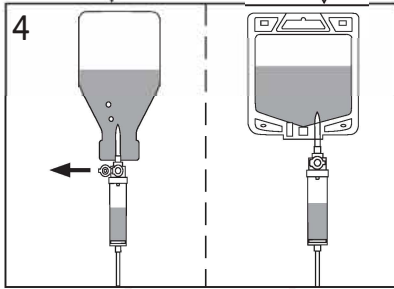
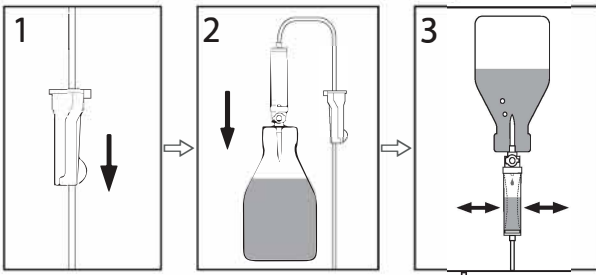
**2018-05**

**TERUFUSION<sup>®</sup>**




**STERILE EO**

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Hình 1

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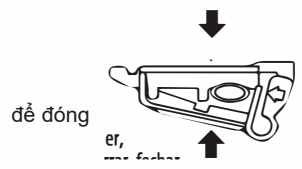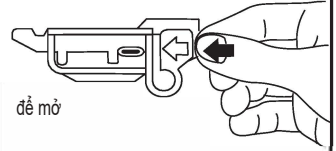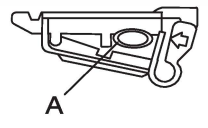**TERUMO MEDICAL PRODUCTS (HANGZHOU) CO., LTD.**

M4-9-5 Hangzhou Economic and Technological Development Zone, 310018 Hangzhou, P.R.CHINA

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C.P. 03900, Distrito Federal, México.

**TERUMO** is a trademark of TERUMO CORPORATION.

® : Registered Trademark

---

A large, bold version of the Terumo logo, featuring the stylized symbol followed by the word 'TERUMO' in a bold, sans-serif font.

---

\*TS69J001C2\*

**TERUMO**

2018-05

**TERUFUSION<sup>®</sup>**

CE 0123

**SOLUTION ADMINISTRATION SET for infusion pump**  
**TUBULURE pour pompe à perfusion**  
**INFUSIONSGERÄT für Infusionspumpen**  
**EQUIPO DE INFUSIÓN para bomba de infusión**  
**EQUIPO DE INFUSÃO INTRAVENOSA para uso em bomba de infusão**  
**SET PER SOMMINISTRAZIONE DI SOLUZIONI con pompa per infusione**  
**INFUSSET voor infuus pomp**  
**INFUSIONSSÆT til infusionspumpe**  
**INFUSJONSSETT for infusionspumpe**  
**SÚPRAVA NA PODÁVANIE ROZTOKOV pre infúznú pumpu**  
**INFÚZIÓS SZERELÉK infúziós pumpákhoz**  
**ZESTAW DO INFUZJI do pomp infuzyjnych**  
**УСТРОЙСТВО ДЛЯ ВЛИВАНИЯ РАСТВОРОВ для инфузионных насосов**  
**НАБОР ЗА ВЪВЕЖДАНЕ НА РАЗТВОР за инфузионна помпа**  
**KOMPLET ZA DAVANJE OTOPINE za infuzijsku pumpu**  
**SET PRO PODÁVÁNÍ ROZTOKU pro infuzní pumpu**  
**LAHUSE MANUSTAMISKOMPLEKT infusioonipumbale**  
**LIUKSEN ANNOTELUSETTI infuusiopumpulle**  
**ΣΕΤ ΧΟΡΗΓΗΣΗΣ ΔΙΑΛΥΜΑΤΟΣ για αντλία έγχυσης**  
**SET ZA DAVANJE INFUZIONOG RASTVORA za pumpe za infuziju**  
**ŠĪĻDUMA IEVADES KOMPLEKTS infūzijas sūkņim**  
**RĪNKINYS TIRPALO DOZAVIMUI naudojant infuzinę pompą**  
**İnfüzyon pompası için ÇÖZELTİ UYGULAMA SETİ**  
**SET DE ADMINISTRARE ȘOLUTIE pentru pompă de perfuzie**  
**INFUZIJSKI SET za infuzijske črpalke**  
**INFUSIONSSET för infusionspump**

**STERILE EO**





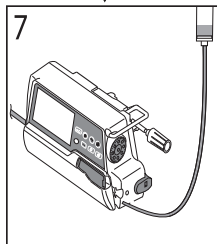
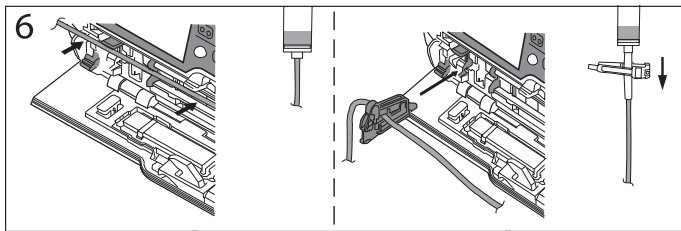
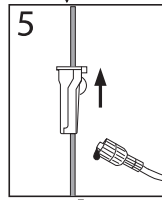
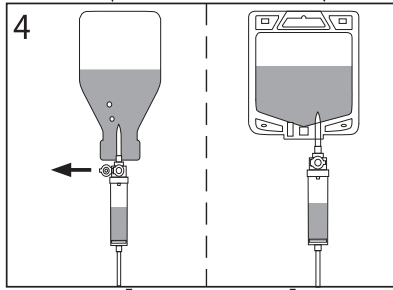
Fig. 1, Abb. 1, Obr. 1, 1. ábra, Rys 1, Рис. 1, Фиг. 1, Slika 1, Joonis 1, Kuva 1, Σχ. 1, 1. att., 1 pav., Şek. 1

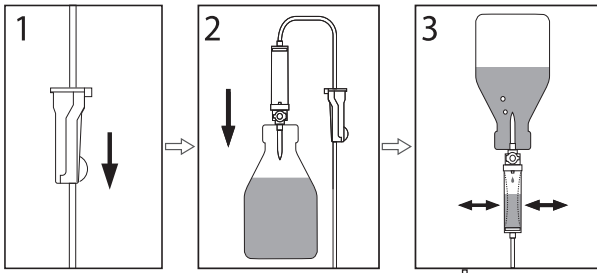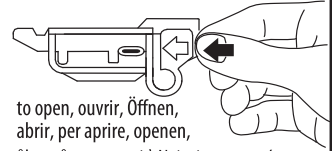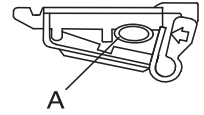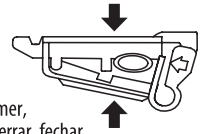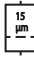
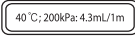
to open, ouvrir, Öffnen, abrir, per aprire, openen, åbne, åpne, otvoriti, Niyitni, otworzyć, Открыть, за да отворите, otvoriti, otevřít, avamiseks, avaa, για άνοιγμα, za otvaranje, atvērt, atidaroma, açmak için, pentru a deschide, opiranje, öppna

to close, fermer, Schließen, cerrar, fechar, per chiudere, sluiten, lukke, zavriet, Zárni, zamknąć, Закрыть, за да затворите, zatvoriti, zavřít, sulgumiseks, sulje, για κλείσιμο, za zatvaranje, aizvērt, uždaroma, kapatmak için, pentru a închide, zapiranje, stänga

## Index of Symbols

					
Does not contain latex	Does not contain DEHP	Does not contain PVC	Non-pyrogenic	Do not use if package is damaged	Length of pump tubing
					
Luer Lock Connector	Luer Connector	Pressure	Anti-Free Flow Clip	20 drops per milliliter	With Airvent
					
For TERUFUSION pumps	Liquid filter with 15 µm pore size	Injection port	Safe for use with Terufusion pumps. Storage volume at 40 °C; 200kPa: 4.3mL/1m		

## PRECAUTIONS

- Do not use if protective caps are loose or displaced.
- Do not use if package is damaged.
- Use immediately after opening the unit packaging.
- Dispose of safely after use to avoid risk of infection.
- Ensure that all connections made with the Terufusion solution administration set are secure.
- Do not turn upside down or shake the drip chamber during administration.
- Disinfect the injection port before use.
- We recommend to switch the tubing section used in the pump after maximum 24h to another position, as the flow rate may deviate from the set value due to deformation of the tubing section in the pump after long period of use.
- For single use only. Do not reuse. Do not resterilize. Do not reprocess. Reprocessing may compromise the sterility, biocompatibility and functional integrity of the device.
- This product has been sterilized by ethylene oxide gas.
- **DO NOT STORE AT EXTREME TEMPERATURES.**
- Do not detach the Anti Free Flow clip (AFF clip) from the administration set.
- Do not slide up or down the AFF clip while closed.
- Make sure to use only TERUFUSION pumps (with AFF function if required).

★ If the AFF clip is available, 1. Make sure that the AFF clip is not closed. 2. Slide the AFF clip down. 3. Make sure that the tube is positioned in the AFF clip as indicated in Fig 1-A, and insert the clip into the dedicated aperture of the infusion pump. Then attach the tube properly to the tube guide slots.

## Caution

Read the instruction manual of the infusion pump before use.