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SURPLUG AD Three-way stopcock

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PVC Free

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TERUMO

TERUMO MEDICAL PRODUCTS (HANGZHOU) CO., LTD.
M4-9-5 Hangzhou Economic and Technological Development Zone, 310018 Hangzhou, P.R.CHINA

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SURPLUG AD Three-way stopcock

Index of Symbols

Does not contain PVC / drug contact component

Does not contain latex

Non-pyrogenic

Do not use if package is damaged

Consult instructions for use

Do not reuse

Manufacturer

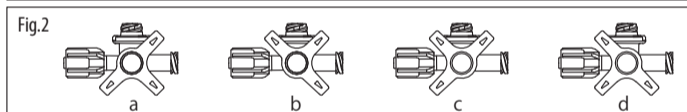
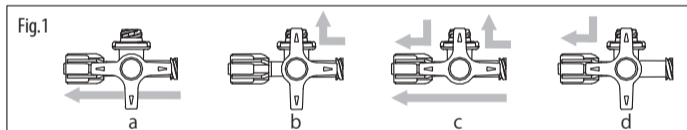
Sterilized using ethylene oxide

Intended Use

SURPLUG AD Three-way stopcock is used as a valve port connected to the fluid infusion set or a similar device for injection. It can also be used for blood sampling and transfusion.
SURPLUG AD Three-way stopcock has the flow pathway to prevent medications from stagnation.

How to Use

- The arrows below show flow pathway. To set stopcock position depending on use. By switching stopcock position, flow pathway is switched.
- To keep the stopcock position as Fig. 1c to prevent medications from stagnation in the flow pathway, even if the SURPLUG AD Three-way stopcock valve port is not used.
- In order to close the flow path in all directions, it is set at the position of the stopcock in Fig. 2.

Warnings

- Disinfect the SURPLUG AD Three-way stopcock valve port before use.
- To apply heparin lock, close the flow pathway of patients side with stopcock before removing the syringe. While applying heparin, do not open the flow pathway that has been closed.

Cautions

- For single use only. Do not reuse.
- Do not resterilize.
- Do not use a needle for injection.
- Do not use if the package is damaged.

Precautions

- Make sure that there are no loose connections before use.
- Set the stopcock into open position and connect to the SURPLUG AD Three-way stopcock valve port (Fig. 1b/Fig. 1c/Fig. 1d).
- When connecting a male luer connector to the SURPLUG AD Three-way stopcock valve port, be sure to insert and remove it straight. Do not connect at an angle or with excessive twisting.
- Do not use if any part of the product is damaged.
- Make sure regularly that there are no breakage, no loose connections or leakage during use.
- Do not apply high-pressure injection such as contrast agent using injector.
- Do not connect a connector with sharp edge to the SURPLUG AD Three-way stopcock valve port.
- When using a connector with bump around the end, a connector with sharp edge or a connector with burr on the edge, make sure that the flow pathway is secured before starting injection, because the SURPLUG AD Three-way stopcock valve port may remain closed.

Storage Method

- Do not store in direct sunlight and high temperature and humidity.

Expiry Date

- Expiration date stated in the outer box.

TERUMO

TERUMO MEDICAL PRODUCTS (HANGZHOU) CO., LTD.
M4-9-5 Hangzhou Economic and Technological Development Zone, 310018 Hangzhou, P.R. CHINA

All brand names are trademarks or registered trademarks of TERUMO CORPORATION and their respective owners.

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© : TERUMO CORPORATION 10/2020