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## **SURFLO I.V Catheter Radiopaque**

### **CONTRAINDICATION**

- **Contraindicated where Magnetic Resonance Imaging (MRI) is anticipated.**

### **PRECAUTIONS**

- **Read Instructions for Use.**
- **Do not use if the package or the product has been damaged or contaminated.**
- **Do not attempt to reinsert a partially or completely withdrawn needle.**
- **Handle with care to avoid needlestick.**
- **Do not re-use or re-sterilize**
- **Dispose of properly after single use to avoid risk of infection**
- **DO NOT STORE AT EXTREME TEMPERATURES AND HUMIDITY. AVOID DIRECT SUNLIGHT.**

### **INSTRUCTIONS FOR USE**

- 1. Open the case with a simple twisting motion to break the seal.**
- 2. Remove the I.V. Catheter straight out of the case using two hands.**
- 3. A box-shaped dent located on the upper surface of the flashback chamber is a needle bevel orientation indicator.**
- 4. After confirming flashback at flashback chamber, advance the catheter and needle together a little further.**
- 5. Advance the catheter into the vein.**
- 6. Holding the catheter hub firmly, withdraw the needle.**
- 7. Connect administration set and secure the devices safely.**

### **INTENDED USE**

- **TERUMO SURFLO IV Catheter is used for the purpose of infusing fluids into patients for various clinical applications. It is inserted into the patient's vascular system for short term use (less than 30 days) to withdraw blood samples, monitor blood pressure, or administer fluids intravenously.**

**CONFIDENTIAL INFORMATION**