



HƯỚNG DẪN SỬ DỤNG

KIM TIÊM VÔ TRÙNG_TERUMO NEEDLE AGANI

KIM TIÊM VÔ TRÙNG

SẢN PHẨM SỬ DỤNG MỘT LẦN

SẢN PHẨM ĐÃ ĐƯỢC TIỆT TRÙNG, KHÔNG CHỨA PYROGENIC

ĐÓNG GÓI: 100 CHIẾC/ HỘP


<CẢNH BÁO>

- Sử dụng ngay sau khi mở bao gói sản phẩm
- Thải bỏ sản phẩm đúng cách ngay sau khi sử dụng để tránh lây nhiễm
- Cần trọng để tránh đầu kim đâm vào tay
- Không thay thế ống tiêm sau khi đã sử dụng
- Sản phẩm sử dụng 1 lần. Được tiệt trùng bằng tia gamma. Không được tái sử dụng hoặc tiệt trùng lại
- Không sử dụng nếu bao gói sản phẩm bị rách hoặc bẩn

<HƯỚNG DẪN SỬ DỤNG>

1. Xé bao gói sản phẩm như hình minh họa
2. Cần thận gắn kim tiêm vào xi lanh
3. Rút nòng kim ra khỏi nắp bảo vệ. Kim đã được tiệt trùng và sẵn sàng sử dụng

Lưu ý: Không cầm vào kim khi không có nắp bảo vệ

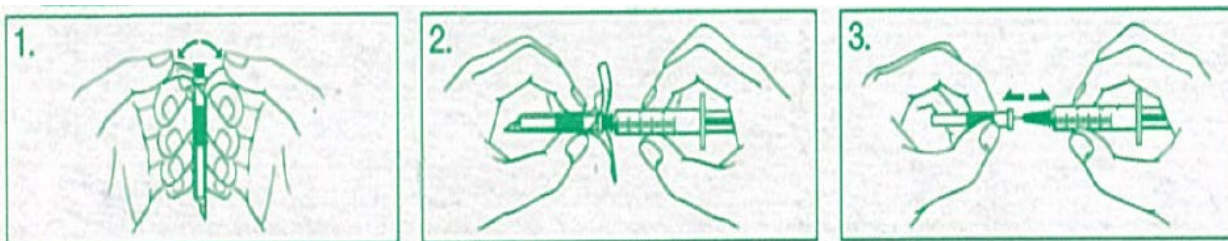

- **Không để sản phẩm ở nơi có ánh sáng mạnh hoặc độ ẩm cao. Tránh tiếp xúc với nước hoặc để trực tiếp dưới ánh nắng mặt trời**

Nhà sản xuất: Zhejiang Kindly Medical Devices Co., Ltd

Phân phối bởi: Terumo Corporation

44-1, 2-chome, Hatagaya, Shibuya-ku, Tokyo 151-0072, Japan



IFU

TERUMO AGANI NEEDLE

TERUMO®

TERUMO NEEDLE **AGANI™**

SINGLE USE
STERILE · NON-PYROGENIC

100 each

TERUMO®

<WARNING/PRECAUTIONS>

WARNING

- Use immediately after opening package. After use, dispose of needle properly to preclude the risk of infection.
- Handle with care to avoid needle stick. Do not replace needle sheath after use.

PRECAUTIONS



- For single use only. Sterilized by gamma radiation. Do not resterilize or reuse.
- Do not use if the unit package or product has been damaged or contaminated.

CAUTION: Federal law (U.S.A) restricts this devices to be sold by or under the order of a physician
NO.252, Yongqiang Road, Yongzhong, Longwan, Wenzhou, Zhejiang Province, P.R.C.

<INSTRUCTIONS FOR USE>

1. Open blister package as illustrated.
2. Attach needle securely to syringe.
3. Pull straight needle sheath to remove. Needle is sterile and ready for use.

CAUTION Avoid handling needle without sheath.

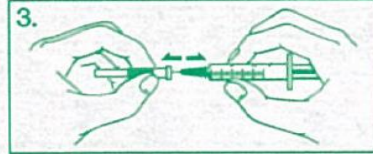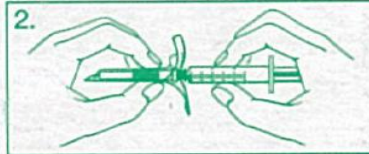

- Do not store at extreme temperatures and humidity. Avoid exposure to water or direct sunlight during storage.

MANUFACTURED BY : **Zhejiang Kindly Medical Devices Co.,Ltd**

252, Yongqiang Road, Yongzhong, Longwan, Wenzhou, Zhejiang, P.R.C

SOLD BY: **TERUMO CORPORATION**

44-1,2-chome, Hatagaya, Shibuya-ku, TOKYO 151-0072, JAPAN