SP69M000C1

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® : Registered Trademark.

SURPLUG is a trademark of TERUMO CORPORATION.

© TERUMO CORPORATION OCTOBER 2010 SP-D-2010-ET-D-01

SURPLUG™ Needle-Free Connector

SURPLUG™ Needle-Free Connector with extension tube

Read carefully before use
Safety and Efficacy Considerations:

Intended Use

SURPLUG is used as a valve port connected to the fluid infusion set or a similar device for injection. It can also be used for blood sampling.

⚠ Warnings

- Use ethanol cotton to disinfect the SURPLUG valve port before starting injection or blood sampling.
- Do not use a needle for injection.
- To apply heparin lock (see “Instruction for Use”), close the clamp between the catheter and SURPLUG before removing the syringe. While applying heparin, do not open the clamp that has been closed.

⚠ Cautions





- While applying injection or collecting blood, ensure that the connection is secured to prevent disconnection. For continuous injection in particular, use the syringe or infusion set with lock connector.
- After connecting the male connector to the SURPLUG, do not give excessive force on the connecting point.
- Perform flushing of the SURPLUG before and after the injection, if necessary.
- Remove air from the SURPLUG, if necessary.

⚠ Precautions





- Read the instruction of the drug and other medical equipments used with SURPLUG before use.
 - Do not apply high-pressure injection such as contrast agent using injector.
 - When using a connector with bump around the end, a connector with sharp edge or a connector with burr on the edge, make sure that the fluid route is secured before starting injection, otherwise the valve port may remain closed.
- <for “SURPLUG Needle-Free Connector with extension tube”>
- When closing the one-touch clamp, make sure that the tube is securely closed.
 - Make sure that you do not damage the tube by pinching it or using any other sharp objects.
 - Do not apply excessive force to the tube or connecting part of the tube, by pulling, pushing or bending the tube.

Instruction for Use





Procedure of Injection (one-shot injection with a syringe):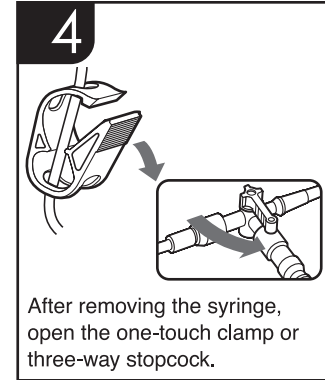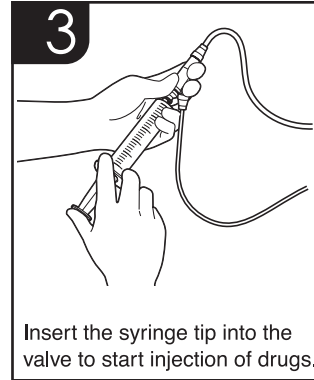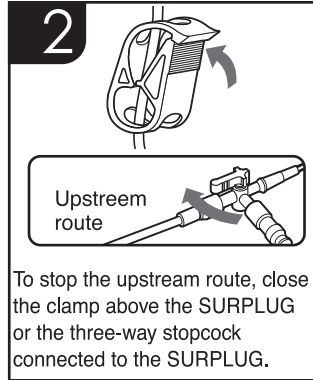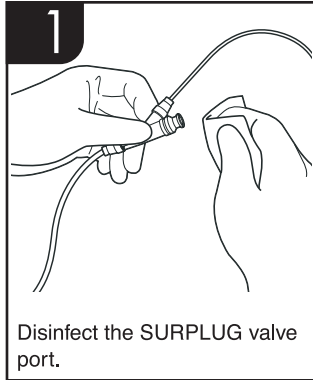

Procedure of Injection (continuous injection with a fluid infusion set):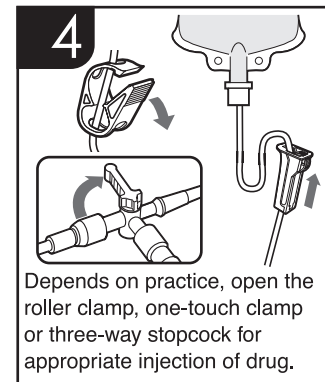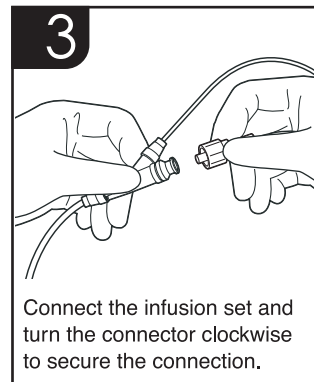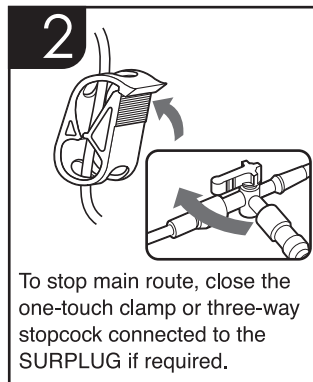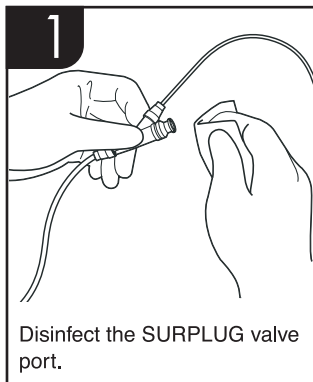

Procedure of Heparin Lock:

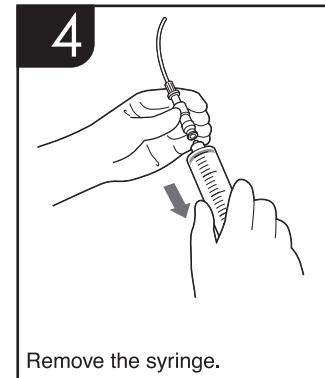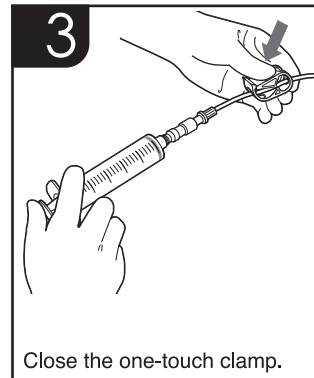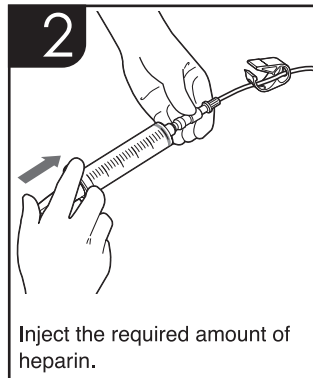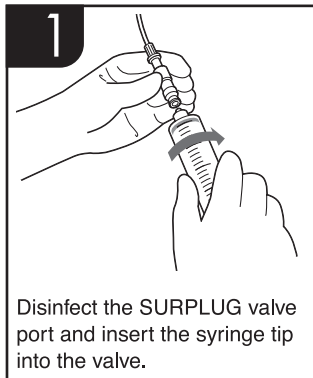Note While applying heparin, do not open the one-touch clamp.

SmartSite[®]

SURPLUG is using SmartSite manufactured by ALARIS Medical Systems, Inc. SmartSite and ALARIS are registered trademarks of ALARIS Medical Systems, Inc. SmartSite is copyright protection technology that is protected by the following patents owned by ALARIS Medical Systems, Inc.: U.S. patents (No. 5676346, No. RE35841, No. 5699821), Australia patent (No. 686942), Japan patent (No. 2984642), New Zealand patent (No. 286445), and more issued or applied in the U.S. and other countries.

PT. TERUMO INDONESIA
Wisma Kyoel Prince 5th Floor
JL. Jend. Sudirman KAV.3
Jakarta 10220, Indonesia
TEL: 62-21-572-4071
FAX: 62-21-572-4072

TERUMO (THAILAND) CO., LTD
#1206, 12th Floor 54 B.B. Building
Asoke Road (Sukhumvit 21), Kwaeng Klongtoeynu, Khet Wattana, Bangkok 10110, Thailand
TEL: 66-2-260-7020
FAX: 66-2-260-7019

**TERUMO (THAILAND) CO., LTD
HO CHI MINH CITY REPRESENTATIVE OFFICE**
HOANG ANH SAFOMEK Office Building,
Room No. 705, 7th FL.,
549-552 Nguyen Tri Phuong Str.,
Ward 14, District 12, Ho Chi Minh City, Vietnam
TEL: 84-8-866-9263
84-8-866-9264
FAX: 84-8-866-9261

**TERUMO (THAILAND) CO., LTD
HANOI REPRESENTATIVE OFFICE**
International Centre 17 Ngo Quyen str.,
Unit 05, 6th Floor Hanoi, Vietnam
TEL: 84-4-936-1643
84-4-936-1644
FAX: 84-4-936-1641

TERUMO KOREA CORPORATION
6F, Shinwon Bldg., 823-14
Yeoksam dong, Kangnam-gu,
Seoul, 135-933, Korea
TEL: 82-2-565-9225
FAX: 82-2-565-9224

TERUMO MARKETING PHILIPPINES INC.
Unit 3203, West Tower
The Philippines Stock Exchange Centre
Exchange Road, Ortigas Center,
Pasig City, Philippines
TEL: 63-2-632-1674
63-2-632-1675
63-2-632-6147
FAX: 63-2-632-7966

TERUMO[®]

TERUMO MEDICAL PRODUCTS HANGZHOU CO., LTD. Hangzhou, China 310018
TERUMO CORPORATION TOKYO, 151-0072 JAPAN

® : Registered Trademark.

SURPLUG is a trademark of TERUMO CORPORATION.

© TERUMO CORPORATION OCTOBER 2010 SP-CH-2010-ET-D-01