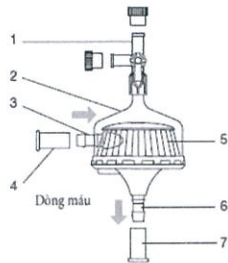
CAPIOX® ARTERIAL FILTER

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Indication for use

The CAPIOX Arterial Filter is intended to filter nonbiologic particles and emboli and to facilitate air bubble removal from the blood flowing through a cardiopulmonary bypass circuit for up to 6 hours. The CAPIOX Arterial Filter creates a spiral blood flow, allowing air bubbles to be removed efficiently.

General Precautions

- This device should only be used by properly trained and qualified personnel.
- This device is sterile and non-pyrogenic in the unopened, undamaged package. Inspect the device and package carefully. Do not use if the package and/or device is damaged, or if the caps are not in place.
- Use immediately after opening the unit packaging.
- Dispose of safely after single use to avoid risk of infection.
- This product has been sterilized by ethylene oxide gas.
- For single use only. Do not reuse. Do not resterilize. Do not reprocess. Reprocessing may compromise the sterility, biocompatibility and functional integrity of the device.
- Do not use solvents such as alcohol, ether, cyclohexanone, etc. These solvents may cause damage if used in or on the device.
- Adequate heparinization of the blood is required to prevent it from clotting in the system.
- Use aseptic technique in all procedures.
- Pressure at the blood inlet of the CAPIOX Arterial Filter should not exceed 101kPa (760mmHg). Pressures greater than 101kPa (760mmHg) may cause leaks or damage to the device.
- Do not add excessive shock to the filter such as tapping with forceps during priming or bypass, as this could damage the product.
- Avoid exposure to water, direct sunlight, extreme temperature, or high humidity during storage.
- Rx ONLY** CAUTION: Federal law (U.S.A.) restricts this device to sale by or on the order of a physician.

Operation

The CAPIOX Arterial Filter is designed for use in the arterial line of the blood circuit, with an air vent line connected to the Cardiomy Reservoir. This device should be used with an appropriate holder supplied by TERUMO. (See specifications)

A. Set-up

1. Remove the CAPIOX Arterial Filter from the package and examine it for defects.
- CAUTION**
- Do not use if the package and/or device is damaged (e.g. cracked), or if any of the port caps are off.
2. Connect the CAPIOX Arterial Filter into the arterial line. Filters should be installed into cardiopulmonary systems with a bypass line to allow easy priming and to permit bypassing the filter if necessary during

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- cardiopulmonary bypass. (See Fig.1)
- NOTE:** The blood inlet port is fitted with a blue cap, and the blood outlet port with a red cap.
3. Attach a 3-way stopcock to the Air Vent port.
 4. Attach one end of a male-male extension tube to the Air Vent port and connect the other end to a luer port on the Cardiomy Reservoir. Close the stopcock to the filter.
 - NOTE:** It is recommended that the tube has a one way valve in-line to prevent backflow.
 5. Fit the filter vertically in the holder with the blood outlet port at the bottom.
 6. Band all connections.

B. Priming

- NOTE:** It is recommended to flush with CO₂ for approximately 5 minutes for easier priming.
1. Clamp the blood lines close to the blood inlet and outlet ports of the CAPIOX Arterial Filter. Prime the circuit through the bypass line. (See Fig. 2) Stop the pump.
 2. Move the outlet clamp to a position downstream of the bypass line, and open the Air Vent port. Start the pump slowly, and fill the filter. Tap the filter gently to remove any air. (See Fig. 3)
 3. Close the stopcock to the filter, and stop the pump.
 4. Start the pump. Initiate circulation at approximately 1L/min. Open the Air Vent port to remove any air. (See Fig. 5)
 5. Close the stopcock to the filter, and stop the pump. Remove the filter from its holder and turn it upside down. Run the pump slowly while gently tapping the filter to remove any air. (See Fig. 6)
 6. Return the filter to its upright position. When all remaining air is removed, put it back in the holder and stop the pump.
 7. In case there are any remaining air bubbles, repeat the priming procedure from step 4 to 6 until all air bubbles are eliminated. (See Fig. 7)

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CAUTIONS

- To remove air bubbles effectively, the CAPIOX Arterial Filter should be set in its holder with the blood outlet port down.
- The CAPIOX Arterial Filter may be used with the 3-way stopcock either open or closed. If the CAPIOX Arterial Filter is used with the stopcock closed, it should be opened to expel air every time any air is collected in the filter. Using the stopcock in the open position, air can be driven out continuously. The stopcock must be closed prior to stopping the pump to prevent backflow of blood and possible exsanguination of the patient.
- Do not hit the filter during cardiopulmonary bypass.
- The CAPIOX Arterial Filter is designed to operate at blood flow rates within the range of 0 to 2.5L/min.
- During cardiopulmonary bypass, should there be a sudden increase of more than 100mmHg in the pressure drop across the filter, the filter must be bypassed, since such an increase can result in blood damage. (See Fig. 8)

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Specifications

Specifications	Code No.: CX*AF02
Materials : Housing : Filter Filter pore size	Polycarbonate Polyester 32µm
Port size : Blood Port : Vent Port	1/4" (6.4mm) Female Luer
Priming Volume	40mL
Pressure Drop (Blood Flow)	3.3kPa (25mmHg) (2.5L/min)
Max. Blood Flow	2.5L/min
Appropriate Holder	Code No.: XX*XH041

Limited Warranty

TERUMO Corporation warrants that the product has been manufactured and packaged with reasonable care, and will be free from defects under normal and proper use. TERUMO Corporation will not be liable for any direct or indirect, incidental, special, general or consequential loss, damage, or expense from the use of this product. TERUMO Corporation's sole obligation under this warranty shall be to repair or replace, at its option, any product that it believes was defective at the time of shipment if notice thereof is received before the expiration date indicated on the package. THIS WARRANTY IS THE SOLE AND ONLY WARRANTY OF TERUMO CORPORATION AND IS EXPRESSLY IN LIEU OF AND EXCLUDES ANY OTHER EXPRESSED OR IMPLIED WARRANTY OF ANY KIND, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR ANY PARTICULAR PURPOSE. IN ADDITION, THE REMEDIES SET FORTH HEREIN ARE THE SOLE REMEDIES AVAILABLE TO ANY PERSON OR ENTITY.

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TERUMO CORPORATION
44-1, 2-CHOME, HATAGAYA, SHIBUYA-KU, TOKYO 151-0072, JAPAN MADE IN JAPAN

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ENGLISH

CAPIOX® ARTERIAL FILTER

Read all cautions and instructions carefully prior to use.

Indication for use

The CAPIOX Arterial Filter is intended to filter nonbiologic particles and emboli and to facilitate air bubble removal from the blood flowing through a cardiopulmonary bypass circuit for up to 6 hours. The CAPIOX Arterial Filter creates a spiral blood flow, allowing air bubbles to be removed efficiently. The blood contacting surfaces of the CAPIOX Arterial Filter are coated with a biocompatible polymer that is intended to reduce the level of platelet adhesion that can occur as blood flows through the unit.

General Precautions

- This device should only be used by properly trained and qualified personnel.
 - This device is sterile and non-pyrogenic in the unopened, undamaged package. Inspect the device and package carefully. Do not use if the package and/or device is damaged, or if the caps are not in place.
 - Use immediately after opening the unit packaging.
 - Dispose of safely after single use to avoid risk of infection.
 - This product has been sterilized by ethylene oxide gas.
 - For single use only. Do not reuse. Do not resterilize. Do not reprocess. Reprocessing may compromise the sterility, biocompatibility and functional integrity of the device.
 - Do not use solvents such as alcohol, ether, cyclohexanone, etc. These solvents may cause damage if used in or on the device.
 - Adequate heparinization of the blood is required to prevent it from clotting in the system.
 - This device is coated with the X-Coating polymer. This is not heparin based.
 - Use aseptic technique in all procedures.
 - Pressure at the blood inlet of the CAPIOX Arterial Filter with X-Coating should not exceed 101kPa (760mmHg). Pressures greater than 101kPa (760mmHg) may cause leaks or damage to the device.
 - Do not add excessive shock to the filter such as tapping with forceps during priming or bypass, as this could damage the product.
 - Avoid exposure to water, direct sunlight, extreme temperature, or high humidity during storage.
- [Rx ONLY]** CAUTION: Federal law (U.S.A.) restricts this device to sale by or on the order of a physician.

Operation

The CAPIOX Arterial Filter is designed for use in the arterial line of the blood circuit, with an air vent line connected to the Cardiotomy Reservoir. This device should be used with an appropriate holder supplied by TERUMO. (See specifications)

A. Set-up

- Remove the CAPIOX Arterial Filter from the package and examine it for

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defects.

CAUTION

Do not use if the package and/or device is damaged (e.g. cracked), or if any of the port caps are off.

- Connect the CAPIOX Arterial Filter into the arterial line. Filters should be installed into cardiopulmonary systems with a bypass line to allow easy priming and to permit bypassing the filter if necessary during cardiopulmonary bypass. (See Fig.1)
NOTE: The blood inlet port is fitted with a blue cap, and the blood outlet port with a red cap.
- Attach a 3-way stopcock to the Air Vent port.
- Attach one end of a male-male extension tube to the Air Vent port and connect the other end to a luer port on the Cardiotomy Reservoir. Close the stopcock to the filter.
NOTE: It is recommended that the tube has a one way valve in-line to prevent backflow.
- Fit the filter vertically in the holder with the blood outlet port at the bottom.
- Band all connections.

B. Priming

NOTE: It is recommended to flush with CO₂ for approximately 5 minutes for easier priming.

- Clamp the blood lines close to the blood inlet and outlet ports of the CAPIOX Arterial Filter. Prime the circuit through the bypass line. (See Fig. 2) Stop the pump.
- Move the outlet clamp to a position downstream of the bypass line, and open the Air Vent port. Start the pump slowly, and fill the filter. Tap the filter gently to remove any air. (See Fig. 3)
- Close the stopcock to the filter, and stop the pump. Remove the filter from its holder and turn it upside down. Run the pump slowly while gently tapping the filter to remove any air. (See Fig. 6)
- Return the filter to its upright position. When all remaining air is removed, put it back in the holder and stop the pump.
- In case there are any remaining air bubbles, repeat the priming procedure from step 4 to 6 until all air bubbles are eliminated. (See Fig. 7)

C. During Cardiopulmonary Bypass

During cardiopulmonary bypass, the following cautions should be observed.

CAUTIONS

- To remove air bubbles effectively, the CAPIOX Arterial Filter should be set in its holder with the blood outlet port down.
- The CAPIOX Arterial Filter may be used with the 3-way stopcock either open or closed. If the CAPIOX Arterial Filter is used with the stopcock closed, it should be opened to expel air every time any air is collected in

the filter. Using the stopcock in the open position, air can be driven out continuously. The stopcock must be closed prior to stopping the pump to prevent backflow of blood and possible exsanguination of the patient.

- Do not hit the filter during cardiopulmonary bypass.
- The CAPIOX Arterial Filter is designed to operate at blood flow rates within the range of 0.5 to 7L/min.
- During cardiopulmonary bypass, should there be a sudden increase of more than 100mmHg in the pressure drop across the filter, the filter must be bypassed, since such an increase can result in blood damage. (See Fig. 8)

Specifications

Specifications	Code No.: CX*AF125X
Materials : Housing : Filter Filter pore size	Polycarbonate Polyester 37µm
Port size : Blood Port : Vent Port	3/8" (9.5mm) Female Luer
Priming Volume	125mL
Pressure Drop (Blood Flow)	4.8kPa (36mmHg) (7.0L/min)
Blood Flow Range	0.5-7.0L/min
Appropriate Holder	Code No.: XX*XH041

Limited Warranty

TERUMO Corporation warrants that the product has been manufactured and packaged with reasonable care, and will be free from defects under normal and proper use.

TERUMO Corporation will not be liable for any direct or indirect, incidental, special, general or consequential loss, damage, or expense from the use of this product. TERUMO Corporation's sole obligation under this warranty shall be to repair or replace, at its option, any product that it believes was defective at the time of shipment if notice thereof is received before the expiration date indicated on the package.

THIS WARRANTY IS THE SOLE AND ONLY WARRANTY OF TERUMO CORPORATION AND IS EXPRESSLY IN LIEU OF AND EXCLUDES ANY OTHER EXPRESSED OR IMPLIED WARRANTY OF ANY KIND, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR ANY PARTICULAR PURPOSE. IN ADDITION, THE REMEDIES SET FORTH HEREIN ARE THE SOLE REMEDIES AVAILABLE TO ANY PERSON OR ENTITY.

No agent, employee or representative of TERUMO Corporation has any authority to change, amend or alter the foregoing or assume or bind TERUMO Corporation to any additional liability or responsibility in connection with this product.

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© : Marque Enregistrée Xcoating est une marque de TERUMO CORPORATION.
® : Registered Trademark Xcoating is a trademark of TERUMO CORPORATION.